
Bedienungsanleitung
User manual
Manual de instrucciones
Mode d'emploi

TSE-Easy 10

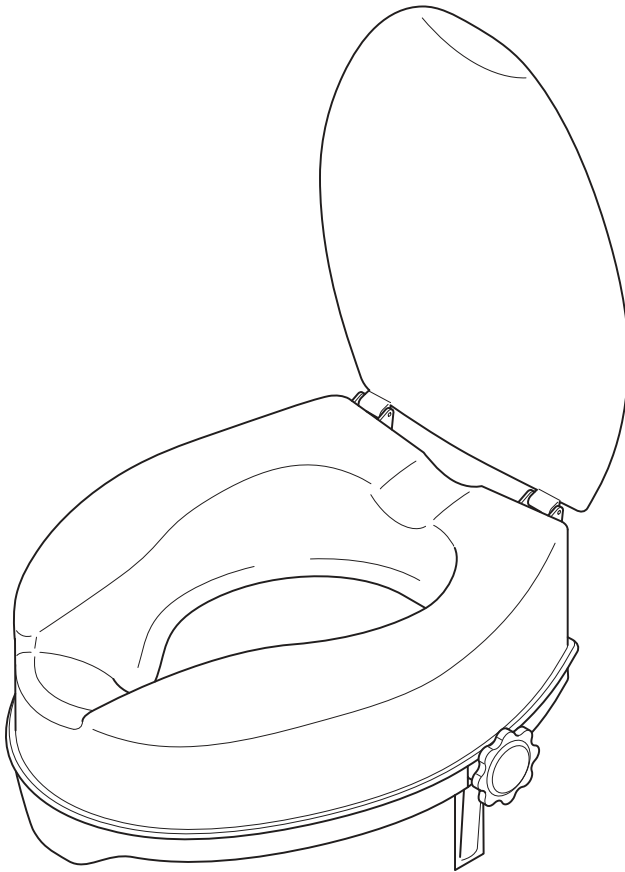Enjoy mobility. 

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Contents

| | | | |
|---------------------------------------|-----------|--|-----------|
| 1. Introduction | 12 | 3.2. Product image/scope of delivery | 16 |
| 2. Safety information | 13 | 3.3. Product's identification label | 16 |
| 2.1. Labels and symbols | 13 | 4. Assembling the toilet seat raiser | 17 |
| 2.2. Intended use | 13 | 5. Mounting the toilet seat raiser | 18 |
| 2.3. Indications | 13 | 6. Usage and care | 19 |
| 2.4. Contraindications | 13 | 7. Maintenance and repair | 19 |
| 2.5. General safety information | 14 | 8. Transfer/Re-use | 20 |
| 2.6. Special safety information | 14 | 9. Storage and shipping | 20 |
| 2.7. Service life | 15 | 10. Disposal | 20 |
| 2.8. Declaration of compliance | 15 | 11. Warranty | 21 |
| 3. Components/Overview | 15 | 11.1. Warranty certificate | 21 |
| 3.1. Technical data | 15 | Explanation of symbols | A1 |

1. Introduction

Dear Customer,

You have chosen a high-quality toilet seat raiser from Bischoff & Bischoff.

Thank you for trusting in us.

This User manual contains all the necessary information for assembling, adjusting, using and looking after the TSE-Easy 10 toilet seat raiser.

Before using your new toilet seat raiser for the first time, please read and follow the information in this User manual carefully,

including all safety information. This manual is an integral and necessary part of the product. Keep the User manual within easy reach and pass it on with the toilet seat raiser when transferring the product.

For individuals who are visually impaired, this document is available as a pdf file on our website www.bischoff-bischoff.com.

2. Safety information

2.1. Labels and symbols

IMPORTANT!

Indicates particularly useful information in a particular context.

You will find the explanations of any additional symbols used in the appendix at the end of this User manual.

2.2. Intended use

A toilet seat raiser supports an independent toilet seat usage caused by physical restrictions. The height of the seat raiser needs to be correctly adjusted to the user.

2.3. Indications

Use of the toilet seat raiser is advisable for individuals with:

- poor body control,
- significant functional disorders in the lower extremities,
- stiffness in the hips and spine,
- handless persons, quadriplegics or individuals with arm disorders with significant disruption to the gripping function.

2.4. Contraindications

Use of the toilet seat raiser is not advisable for individuals with:

- Perceptual disorders,
- Balance disorders,
- No capability to sit,
- for wheelchair users, if no additional safety handles or standing aids are fitted

IMPORTANT!

Despite thorough research and editing, incorrect information may occur in our User manual.

We reserve the right to make technical modifications.

You can find the current version of the User manuals in the download area of our website www.bischoff-bischoff.com.


2.5. General safety information

Notification of incidents: The provider or user must report

1. any malfunction,
 2. any change in the characteristics or performance as well as
 3. any inadequacy in the labelling or instructions for use of a medical product, which has led or could lead to death
- or a severe deterioration in the health condition of a patient, an employee or a third party, must be reported immediately to the national authority responsible for medical products.

2.6. Special safety information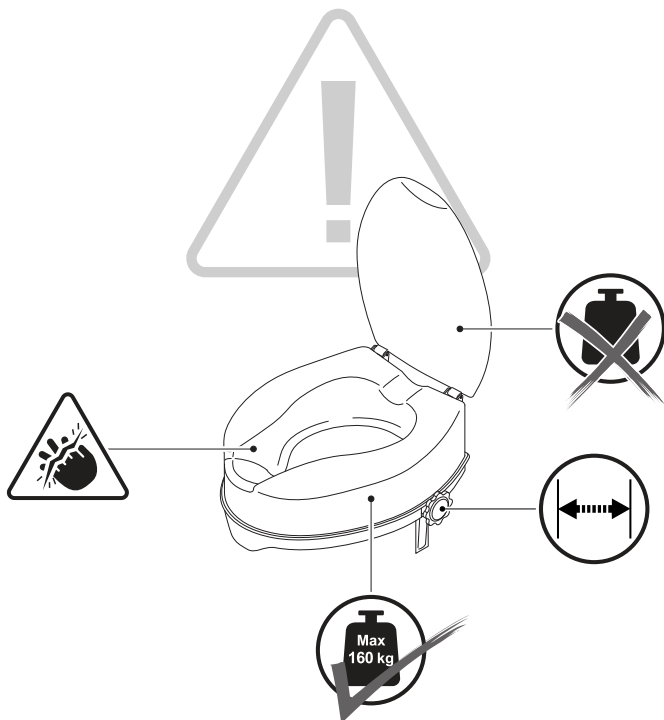

IMPORTANT!

You will find the description of any symbols used in the appendix at the end of this User manual.

2.7. Service life

The expected service life is up to 5 years, if the device is used correctly, and safely, following the care and maintenance instructions.

Usage beyond this period is at own risk and is only allowed if the product is in a condition allowing its safe usage.

2.8. Declaration of compliance

B+B as the product's manufacturer declares that the TSE-Easy 10 toilet seat raiser conforms to the requirements of the

EU Regulation 2017/745 for medical products as well as the 93/42/EEC Directive for medical products.

3. Components/Overview

3.1. Technical data

| | |
|--------------------------------------|------------------------|
| Total length: | 410 mm |
| Total width: | 460 mm |
| Height (with the toilet lid closed): | 155 mm |
| Seat width: | 360 mm |
| Seat depth: | 390 mm |
| Front seat height: | 100 mm |
| | (from the toilet bowl) |
| Hygiene cutout: | 200 x 260 mm |
| Unladen weight: | 1.1 kg |
| Max. Payload: | 160 kg |
| Colour: | white |
| Material: | Polypropylene (PP) |

3.2. Product image/scope of delivery

At receiving the goods, please check that all components have been delivered. The package should contain the following components (Fig. 1):

- ① Toilet lid (1x)
- ② Seat base (1x)
- ③ Hand screw (2x)
- ④ Clamp (2x)
- User manual (1x)
- Packing

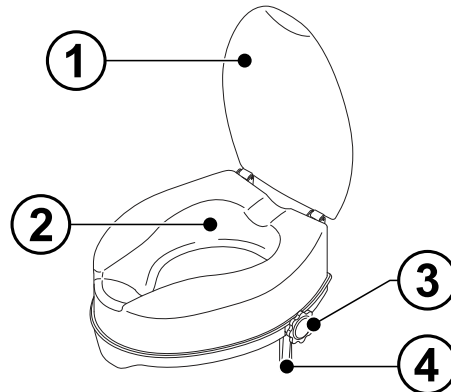
Fig. 1

3.3. Product's identification label


The product's identification label (Fig. 2) is located on the rear of the toilet seat raiser

- Ⓐ Manufacturer's logo
- Ⓑ Manufacturer information
- Ⓒ Date of manufacture
- Ⓓ Model name
- Ⓔ Catalogue number
- Ⓕ Medical product in compliance with EU Regulation 2017/745
- Ⓖ CE mark
- Ⓗ Read the User manual!
- Ⓘ Maximum payload
- Ⓝ Unique Device Identification (UDI)
- Ⓚ Serial number


Fig. 2

4. Assembling the toilet seat raiser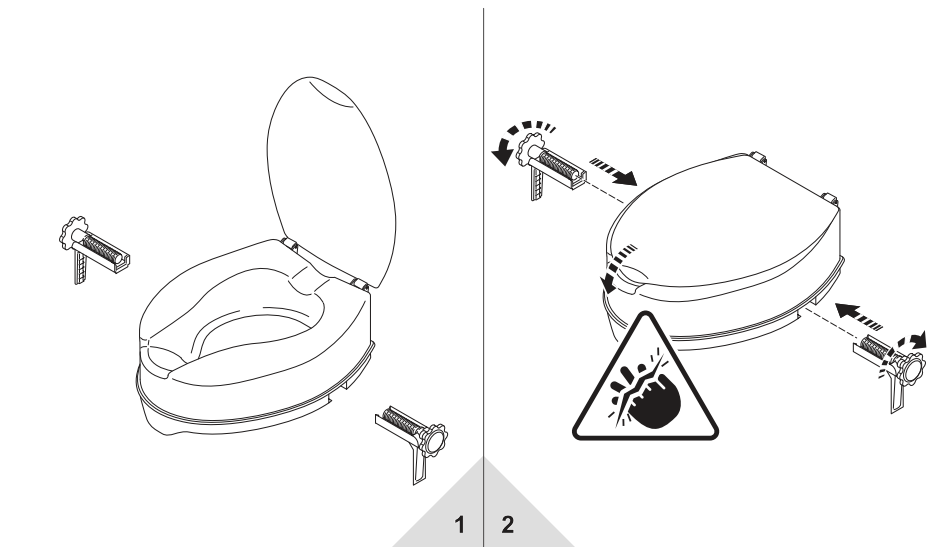


IMPORTANT!
Remove by following the instructions in reverse order.

5. Mounting the toilet seat raiser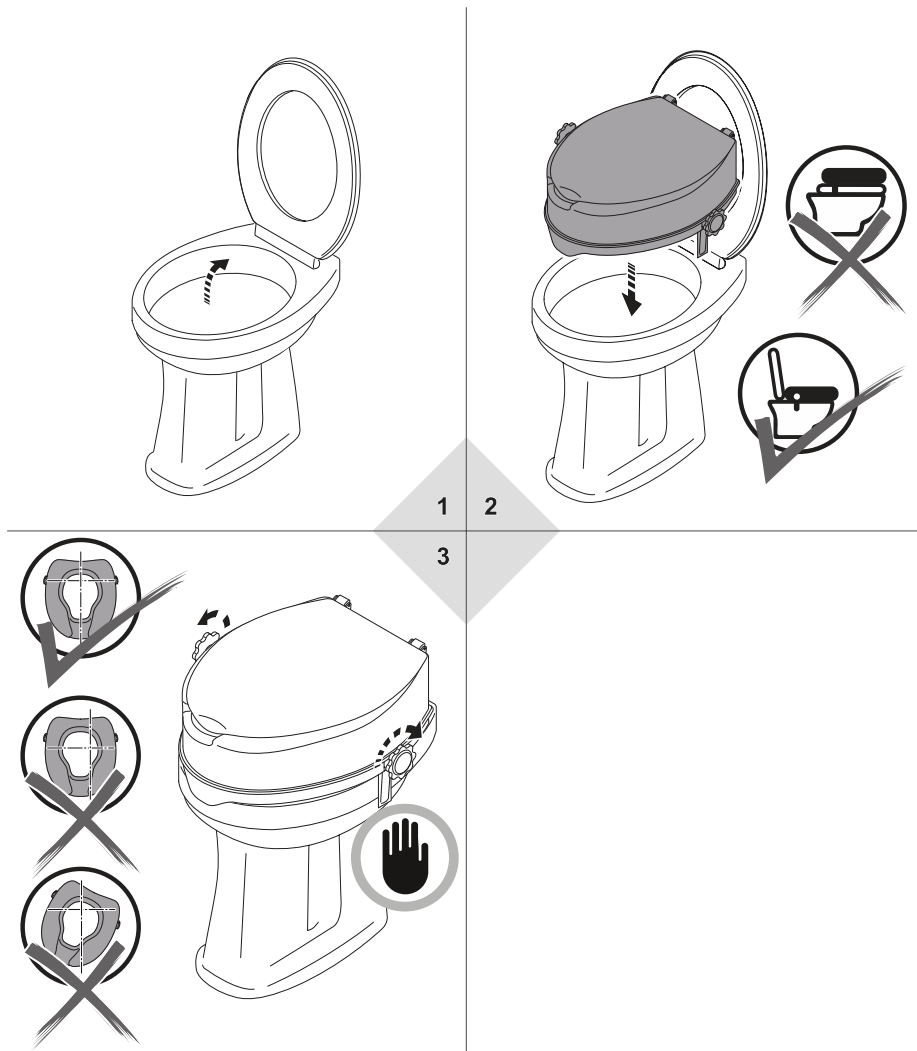

i IMPORTANT!
Remove the toilet seat raiser by following the instructions in reverse order.

6. Usage and care

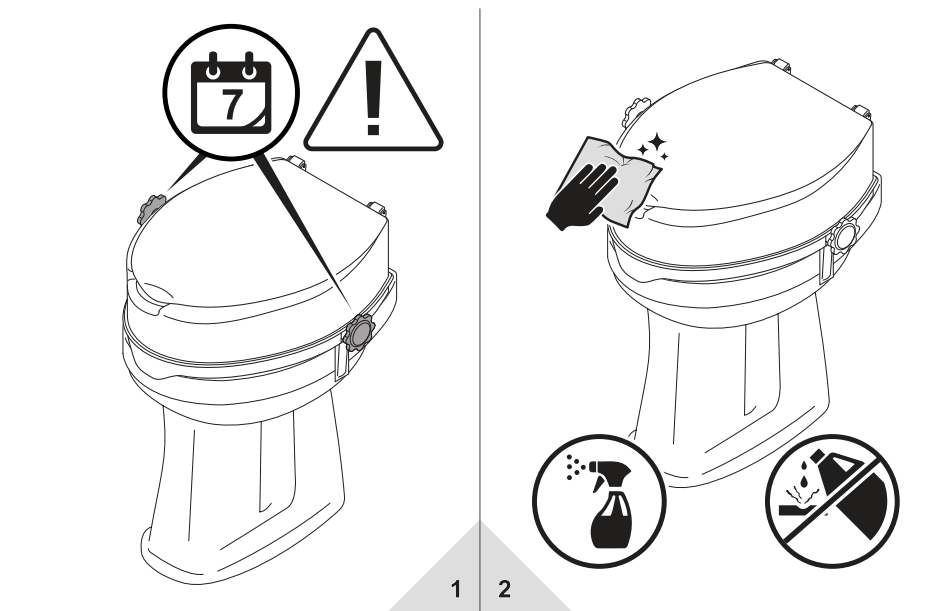7. Maintenance and repair

Your toilet seat raiser is made from high-quality materials for long-term, comfortable use. We carry out in-depth testing of all our products before they leave our factory. However if you should find that your toilet seat raiser has a fault, you should no longer use it. Contact the health-care professional who supplied the product to you to have the product repaired. He will be happy to help you.

Furthermore, we recommend that the toilet seat raiser is serviced at least once a year. Your dealer will receive a maintenance

plan from us together with further documents which are necessary to carry out an inspection.

Only authorised partners (medical supply stores or dealers) are permitted to carry out repairs and maintenance work.

If you have any questions about maintenance and repair, our Customer Service will be happy to help on the free phone number +49 (0)700 6000 7070.

8. Transfer/Re-use

Your toilet seat raiser is designed for re-use. Before passing the toilet seat raiser to another user, it needs to be serviced and disinfected. When the toilet seat raiser is passed to another user, please also hand over this User manual and the address of your dealer.

9. Storage and shipping

Before you store or ship the product, remove all parts which have been plugged in and are not fixed parts. First, pack these individually and then pack them all together in a suitable cardboard box. Where possible, retain and use the original packaging for this purpose. In this way the product has the highest level of protection against environmental influences during storage or transportation.

10. Disposal

Please contact your dealer for professional disposal of the product.

If you would like to dispose of the product yourself, please observe the local municipal regulations!

11. Warranty

The warranty covers all defects which are proven to be due to faulty material or manufacturing. In the event of a claim, the fully completed warranty certificate must be submitted with the product. B+B provides the legal warranty of 2 years.

User manual as well as not carrying out maintenance work properly and/or making technical modifications and additions (attachments) without the agreement of Bischoff & Bischoff GmbH will lead to loss of warranty and the manufacturer's liability.

Failure to observe the instructions in the

11.1. Warranty certificate

Product: TSE-Easy 10 Toilet seat raiser

Model number:

Serial number: (*)

(*) To be filled in by the dealer.
The product's identification label is located on the back of the toilet seat raiser.

Dealer:

Date and stamp

(Pay attention to our general terms and conditions.)

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B+B France S.A.R.L.