Bedienungsanleitung
Instruction manual
Manual de instrucciones
Mode d'emploi

TS-Care


DE

EN

ES

FR

Enjoy mobility.

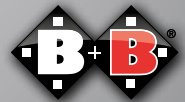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1.	Introduction	19
1.1.	Labels and symbols	19
2.	Important safety information	20
2.1.	Indications	20
2.2.	Contraindications	21
2.3.	Service life	21
2.4.	Intended use	21
2.5.	Declaration of conformity	21
2.6.	Liability	22
3.	Product and scope of delivery	23
3.1.	Checking the delivered goods	23
3.2.	Scope of delivery	23
3.3.	Product's identification label and serial number	24
3.4.	Overview	25
4.	Assembly/Adjustments	26
4.1.	Seat	26
4.2.	Using the toilet	27
5.	Storage/shipping	27
6.	Repairs	27
7.	Technical data	28
8.	Maintenance instructions for the user	30
8.1.	Cleaning	30
8.2.	Disinfection	30
9.	Maintenance	30
10.	Disposal	31
11.	Passing the commode chair on to another user	31
12.	Warranty	32

1. Introduction

Dear Customer,
 you have chosen a high-quality shower aid from Bischoff & Bischoff. Thank you for trusting in us. This User manual contains all the necessary information for assembling, adjusting, using and maintaining your product. Before using your product for the first time, please read and follow the information in this User manual carefully, including all safety information. This manual is an integral and necessary part of the product. Keep the User manual within easy reach and pass it on when transferring the product. For individuals who are visually impaired, this document is available as a pdf file on our website www.bischoff-bischoff.com.

1.1. Labels and symbols

Important! Indicates particularly useful information in the relevant context.

Attention! Indicates particular safety-related Notes. Observe the User manual!

2. Important safety information

- Max. User weight TS-Care 120 kg.
- Only use the commode chair indoors and with the help of an attendant!
- When setting up and folding there is a risk of pinching.
- The seat has been designed for being used as a seat only.
- Avoid excessive sideways leaning Risk of tipping over!
- Make sure that the backrest screws are tight!
- Secure the commode chair against unauthorised use!
- Notification of incidents. The provider or user must report
 1. Any malfunction
 2. Any change in the characteristics or performance as well as
 3. Any inadequacy in the labelling or instructions for use of a medical product, which has led or could lead to death or a severe deterioration in the health condition of a patient, an employee or a third party, must be reported immediately to your appropriate state authority responsible for medical products. They will pass the report on to the authority responsible for the provider immediately and will also inform the manufacturer and the authority responsible for the manufacturer.

2.1. Indications

Use of the commode chair is advisable if you have difficulty standing up or remaining standing due to

- Damage to the functionality of the lower
- extremities with limited
- or failing mobility and
- muscle strength (e.g. loss of strength through
- malignant diseases, chronic,
- advanced cardiovascular and
- pulmonary diseases) and/or
- Damage to the vestibular functions or tendency to fall due to different genesis

- with, at the same time, sufficiently stable trunk posture and sufficient residual functionality of the upper extremities.

2.2. Contraindications

The commode chair is not suitable for individuals with:

- No capability to sit
- severe limitations of cognitive ability
- Perceptual disorders
- Balance disorders

2.3. Service life

The expected service life is up to 5 years, if it is used correctly, and safely, following the care and maintenance instructions. Usage beyond this period is at own risk and is only allowed if the commode chair is in a condition allowing its safe usage.

2.4. Intended use

Use of the commode chair increases independence through use of the toilet and shower. The maximum payload of 120 kg must not be exceeded. The seat must not be used as a climbing aid or for other similar purposes.

2.5. Declaration of conformity

We, Bischoff & Bischoff GmbH, declare under sole responsibility, that the TS-Care commode chair meets all requirements of MDR 2017/745, which are applicable.

2.6. Liability

We can only provide a warranty if the product is used under the conditions specified and for the intended purposes. Modifications, enhancements, repairs and maintenance work is carried out only by those individuals who have been authorised by us to do so, and the commode chair is used in accordance with all the instructions for use.

3. Product and scope of delivery

3.1. Checking the delivered goods

All B+B products undergo an appropriate final inspection at our factory and the CE mark is affixed.

B+B commode chairs are delivered in a special cardboard box. Delivery and induction are carried out by a qualified medical equipment retailer.

After unpacking, retain the cardboard box if possible. It can be used for possible later storage or return of the product.

When sending by train or shipping company, the goods must be checked immediately for transport damage in the presence of the forwarding agent.

If any damage is apparent, contact our Customer service immediately (Tel.: +49 (0)700/6000 7070).

Check that the delivery is complete and undamaged (see Chapter 3.2).

In the case of anomalies or damage, contact our Customer service.

3.2. Scope of delivery

The following is included in the packing:

- Commode chair
- Toilet bucket
- Seat padding
- User manual

3.3. Product's identification label and serial number

The product's identification label and serial number are located on the frame tube.

- A** Manufacturer's logo
- B** Manufacturer information
- C** Attention! - read the User manual
- D** CE mark
- E** Important! - read the User manual
- F** Medical product
- G** Model name
- H** Model number
- I** Date of manufacture
- J** max. payload
- K** UDI item number
- L** Serial number

3.4. Overview

In order to make the handling of your B+B commode chair as easy as possible, we have documented the written explanations in addition to detailed images.

The following images should help you with the naming of the individual components and how they work together (see Fig.).

4. Assembly/Adjustments

Your new B+B commode chair will be delivered in a B+B original cardboard box. In order to avoid transport damage, attachments are delivered separately with the product.

With some workmanship, you can adjust the chair by yourself. Please contact your dealer if you have any questions.

4.1. Seat

Seat height

The height of the TS-Care model can be adjusted to five seat heights. Work in pairs to carry out this work and lie the chair on its back.

- Remove the four circular fixation clips on the frame.
- Now lift the upper part of the frame up until you reach the required height.
- Then refix the four circular fixation clips.

Seat padding

The seat padding has a non-slip underside.

Seat

For better cleaning, you can remove the seat padding. Pull the front seat edge upward out of the support.

- Lift the toilet seat up.

Put the toilet seat back down:

- The seat must click into place in the three locking clamps.

4.2. Using the toilet

Using the toilet bucket

In order to remove the toilet bucket from the commode chair, pull it out of the guide rails from the back.

Use on the toilet

With the commode chair you can use the toilet without the toilet bucket. To do this, first remove the toilet bucket as described above. You can now move your TS-Care over any standard toilet bowl.

Lifting points

You can lift the commode chair using the metal frame, in order to lift it over obstacles or up steps. Avoid lifting the commode by seat and backrest.

5. Storage/shipping

If the chair is to be stored or shipped, all loose parts which are not fixed must be removed and packed individually. The individually packed parts can then be packed together in a larger cardboard box. We recommend that you keep the original packing and store it for these purposes, so that it is available when needed. Your chair is then optimally protected against environmental influences during storage or transportation.

6. Repairs

Repairs and servicing work must only be carried out by authorised partners (medical supply stores or dealers). To find a partner in your area, please contact our Customer Service on tel.: +49 (0)700/6000 7070.

7. Technical data

Dimensions

Width:	540 mm
Depth:	540 mm
Height:	825 mm - 925 mm
Seat width:	420 mm - 430 mm
Seat depth:	420 mm
Seat height without seat plate:	510 mm - 610 mm
Seat height with seat plate:	555 mm - 655 mm (5 adjustments)
Seat to armrests:	195 mm
Seat to backrest:	340 mm
Hygiene cut-out	260 mm x 224 mm oval

Weights

User weight:	120 kg
Kerb weight:	7.8 kg

Materials

Frame:	Colour: blue powder-coated steel
Seat and backrest:	PVC
Flammability:	flame-retardant

Features

padded seat plate
incl. cover plate and toilet bucket

Operating conditions

Temperature:	0 to 50°C
--------------	-----------

Humidity: 20 to 100 %

Heat resistant up to 125°C

Storage conditions

Temperature: -20 to + 60 °C

Humidity: 45 to 60 %

The dimensions stated are below the manufacturing tolerances of +/- 10 mm

8. Maintenance instructions for the user

8.1. Cleaning

To clean the seat and backrest materials, use warm water with the aid of a sponge and/or a soft brush. For more stubborn stains, you can add a household cleaner to the water.

Note: Under no circumstances should you use aggressive detergents, such as, for example solvents or abrasive cleaners or hard brushes.

Look after plastic parts with a household plastic cleaner. Please follow the product information for the cleaner.

8.2. Disinfection

The commode chair can be disinfected with a household disinfectant. Use approved alcohol-based spray or wipe disinfectants. You can find a list of the approved disinfectants from the Robert-Koch-Institute here: <https://www.rki.de/EN>.

Please follow the B+B care, warranty, hygiene, and maintenance information. Your dealer will be happy to provide these to you.

9. Maintenance

Your commode chair is made from high-quality materials for long-term, comfortable use. We carry out in-depth testing on every product before it leaves our factory. Nevertheless, it is possible that a defect may occur. If you should notice a fault with your commode chair, you must stop using it. Contact the healthcare professional who supplied the product to you to have the product repaired. He will be happy

to help you. Furthermore, we recommend that the commode chair is serviced at least once a year. Your dealer will receive a maintenance plan from us together with further documents which are necessary to carry out an inspection. If you have any questions about how to do this, our Customer Service will be happy to provide help and advice on the free phone number +49 (0)700 6000 7070.

You must check your commode chair regularly in accordance with the maintenance plan and make sure that it is in perfect working order.

If you want to handle the disposal yourself, find out about the regulations in your area from local recycling companies.

10. Disposal

When your commode chair is no longer in use and needs to be disposed of, please contact your dealer.

11. Passing the commode chair on to another user

The commode chair is designed for re-use. Before being passed on to another user, it must be serviced by a dealer and hygienically cleaned. When passing the commode chair onto another user, please remember to hand over this user manual and the address of your dealer.

12. Warranty

B+B provides the legal warranty of 2 years for the TS-Care. The warranty covers all defects of the commode chair which are proven to be due to faulty material or manufacturing. In the event of a claim, the fully completed warranty certificate must be submitted with the product.

Failure to observe the instructions in the User manual as well as not carrying out maintenance work properly and/or making technical modifications and additions (attachments) without the agreement of Bischoff & Bischoff GmbH will lead to loss of warranty and the manufacturer's liability.

Warranty certificate

Product: TS-Care

Serial number:*

* (To be filled in by the dealer)

Dealer:

Date and stamp

(Pay attention to our general terms and conditions.)

*Signs are located on the side frame

Note!

Despite thorough research and editing, incorrect information may occur in our User manual. We reserve the right to make technical modifications. Images may show accessories. You can find the current version of the User manuals in the download area of our website: www.bischoff-bischoff.com

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