Bedienungsanleitung
Instruction manual
Manual de instrucciones
Mode d'emploi

VFG

DE

EN

ES

FR

Enjoy mobility.

Inhalt

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Contents

| | | |
|------|--------------------------------|----|
| 1. | Preface | 20 |
| 1.1. | Signs and Symbols | 20 |
| 2. | Important safety instructions | 20 |
| 2.1. | General safety notes | 20 |
| 2.2. | Indications | 22 |
| 2.3. | Contraindications | 22 |
| 2.4. | Declaration of conformity | 22 |
| 2.5. | Responsibility | 22 |
| 2.6. | Useful life | 22 |
| 3. | Purpose | 23 |
| 4. | Product and delivery overview | 23 |
| 4.1. | Delivery Inspection | 23 |
| 4.2. | Scope of delivery | 23 |
| 4.3. | Overview | 24 |
| 4.4. | Type plate and serial number | 25 |
| 5. | Use of the helper | 26 |
| 5.1. | Adjust stick height | 26 |
| 5.2. | Application of the helper | 26 |
| 6. | Technical data | 27 |
| 7. | Care instructions for the user | 28 |
| 7.1. | Cleaning and disinfection | 28 |
| 8. | Repair | 28 |
| 9. | Maintenance instructions | 28 |
| 10. | Passing on the walking aid | 29 |
| 11. | Storage / Shipping | 29 |
| 12. | Disposal | 29 |
| 13. | Warranty | 30 |

1. Preface

Dear user,

You have chosen a high-quality walking aid from Bischoff & Bischoff. We thank you for your trust. These operating instructions contain all the necessary information for operating the VFG walking aid.

Your walking aid is designed for both indoor and outdoor use.

Please read and observe the operating instructions carefully before using your new walking aid for the first time. This is an integral and necessary part of the walking aid. Keep the operating instructions handy and give them to the user when passing on the walking aid.

For visually impaired users, this document is accessible as a PDF file from our website www.bischoff-bischoff.com.

1.1. Signs and Symbols

Important! This is especially useful information on a given subject.

Warning! These are especially safety-relevant instructions. Follow the directions in the manual!

2. Important safety instructions

2.1. General safety notes

- To avoid falls and dangerous situations, first practise using your new walking aid on level, manageable terrain.
- There is a danger of trapping body parts when adjusting the walking aid! Avoid collisions with other objects of any kind.
- Do not hang bags or other objects on the walking aid.
- When walking, the walking aid must be used with the less extended feet pointing to the side of the user, otherwise there is a risk of tripping.

- Make sure that the four-legged support is always placed vertically and not too and not too far away from your body.
- All four feet of the walker must have a firm grip before you move forward with small steps.

Always place the four-footed walking aid vertically on the floor and not too far away from the body. Only when all four feet of the walking aid have a firm grip, move forward with small steps.

- Observe the storage and operating conditions (see chapter 5).
- Reporting of incidents.

If you, as the operator, user or their relatives, notice possible defects or functional restrictions, please contact an authorized specialist dealer immediately. Operators, users or their relatives should inform the dealer who provided you with the product about suspected serious incidents that directly or indirectly had or could have one of the following consequences and by which they are affected. You can also report this to the competent higher federal authority.

- the death of a patient, user or other person,
- the temporary or permanent serious deterioration in the state of health of a patient, user or other person,
- a serious public health hazard.

Protect the walking aid from sunlight and heat to avoid burns from heated components.

Only use the walking aid on firm, level ground.

2.2. Indications

The provision of a walking aid is suitable for people with limited muscle strength and/or balance.

2.3. Contraindications

The provision of a walking aid is unsuitable for people with:

- Joint damage / joint contractures
- Perceptual disorders
- severe balance disorders
- reduced and insufficient vision
- severe limitations of cognitive abilities

2.4. Declaration of conformity

We as Bischoff & Bischoff GmbH declare under our sole responsibility that the walking aid VFG complies with all requirements of MDR 2017/745 that are applicable.

2.5. Responsibility

We shall only provide a warranty if

- the product is used under the specified conditions and for the intended purpose,
- modifications, extensions, repairs and maintenance work are only carried out by persons who have been authorised by us to do so, and
- the walking aid is used in compliance with all instructions for use.

2.6. Useful life

The expected product life is up to four years. The prerequisites for this are the intended use as well as compliance with the safety, care and maintenance instructions in accordance with these operating instructions.

3. Purpose

The walking aid is designed exclusively to support adults with limited mobility. people with the indications (see chapter 2.2 and 2.3). The walking aid can be used on level ground indoors and outdoors. The maximum user weight of 100 kg must not be exceeded. Please note: We can only provide a warranty if the product is used under the specified conditions and for the intended purposes.

4. Product and delivery overview

4.1. Delivery Inspection

All products of Bischoff & Bischoff GmbH are subjected to a proper final inspection in our company and provided with the CE mark (see chapter 4.4).

The walking aid is delivered in a cardboard box. After unpacking, keep the carton if possible. It is used for possible later storage or return of the product.

Delivery and instruction are usually carried out by qualified medical supply retailers.

In the case of shipment by rail or forwarding agent, the goods must be checked immediately for transport damage in the presence of the deliverer.

Check the contents for completeness (see chapter 4.2) and intactness. In case of irregularities or damage, contact our customer service. You will find the contact details on the back of these operating instructions.

4.2. Scope of delivery


Your new Bischoff & Bischoff walking frame is delivered fully assembled in a Bischoff & Bischoff original cardboard box.

After receiving the goods, please immediately check the contents for completeness. The contents consist of:

- Repackaging,
- Walking aid (pre-assembled)

4.3. Overview


- 1 Handle, anatomically shaped, can be used on the left and right
- 2 Telescopic tube
- 3 Spring pin for height adjustment
- 4 Type plate
- 5 Rubber foot anti-slip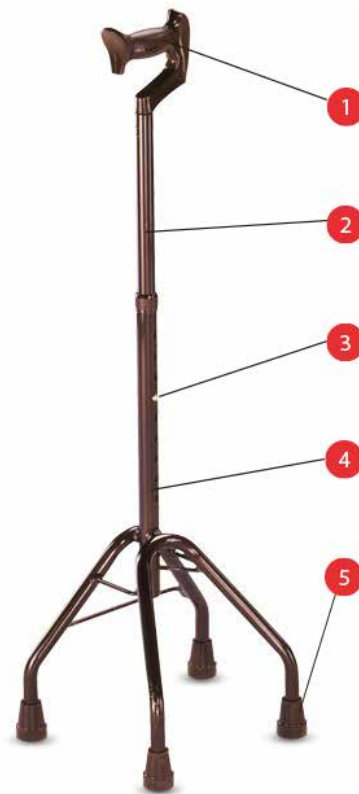

4.4. Type plate and serial number

The type plate (Fig. 1) with the serial number is located on one tube of the walker.

- A Manufacturer logo
- B Manufacturer
- C information
- D Attention! - Observe instructions for use
- E CE mark
- F MD Medical device
- G Important! - Follow the instructions for use Model
- H designation
- I Model number
- J Max. user weight
- K Production date UDI
- L number Serial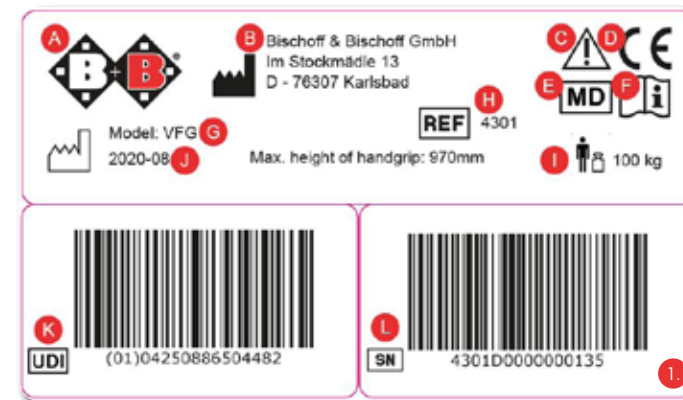

5. Use of the helper

5.1. Adjust stick height


- Press the spring pins on both sides and bring the walking aid into the orthopaedically correct position.
- Snap the spring pins back into place on both sides.

Be careful of your fingers when adjusting, there is a danger of getting caught.

Ensure that the plug connections are correctly seated.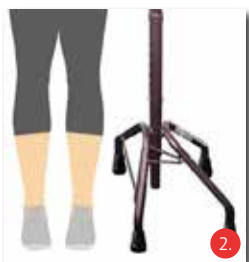

5.2. Application of the helper

- Use the walker only on firm and level ground.
- When walking, the walker should be used with the less extended feet to the side of the user, otherwise there is a risk of tripping, see illustration (Fig. 2).
- Make sure that the four-legged support is always placed vertically and not too and not too far away from your body.
- All four feet of the walker must have a firm grip before you move forward with small steps.

The walker may only be used by persons with a maximum user weight of 100 kg.

6. Technical data

| Dimensions and weights | |
|-------------------------------|-----------------|
| Total height (mm): | 740 - 970 |
| Height adjustment steps | 10 |
| Unladen weight (kg): | 0,8 |
| Max. User weight (kg): | 100 |
| Material | |
| Frame | Bronze anodised |

The dimensions given are subject to manufacturing tolerances of +/- 10mm.

7. Care instructions for the user

7.1. Cleaning and disinfection

- Wipe the frame parts of the walking aid with a damp cloth. For heavier soiling, also use a mild cleaning agent.
- Wash the handle pads with a mild detergent.
- For disinfection, use a disinfectant listed by Verbund für Angewandte Hygiene e.V. (Association for Applied Hygiene).

Do not use a high-pressure cleaner or steam cleaner.

8. Repair

Repairs and maintenance may only be carried out by authorised partners (medical supply stores or specialist dealers). Please contact our customer service by telephone to find a partner near you. You will find the contact details on the back of these operating instructions.

9. Maintenance instructions

The walking aid is maintenance-free.
In the event of malfunctions or defects, hand the walking aid over to your specialist dealer immediately for repair.
The necessary information and documents for repair and maintenance will be made available to the specialist dealer on request. If you have any questions, please contact the Bischoff & Bischoff customer service. The contact details can be found on the back of these operating instructions.

10. Passing on the walking aid

The walking aid is suitable for reuse. Before it is passed on to another user, it must be maintained and hygienically prepared by a specialist dealer. When passing on the walking aid, please remember to give these operating instructions and the address of your specialist dealer to the new user.

11. Storage / Shipping

If the walking aid is to be stored or shipped, it must be packed in a suitable box. It is recommended to keep the original packaging and store it for these purposes so that it is available when needed. Then your walking aid is optimally protected against environmental influences during storage or transport.

For storage of your walking aid, please observe the information in the technical data. To avoid mould growth and damage, the storage location should be as dry as possible and not exposed to direct sunlight.

12. Disposal

If your walking aid is no longer used and is to be disposed of, contact your specialist dealer.

If you want to dispose of the waste yourself, ask local recycling companies about the regulations in your area.

13. Warranty

1. For products supplied by Bischoff & Bischoff, the period of warranty is 24 months from purchase date. The products are delivered free from production and material defects. In case of evidence of material or manufacturing defect, defective parts will be replaced free of charge.

2. This warranty excludes claims for the following reasons:

- a) normal wear and tear, which include but is not limited to the following parts where fitted; Batteries, motor brushes, handles, armrests, upholstery, tyres, brakes, caps, etc.
- b) overloading the product such as, for example, by exceeding the maximum user weight or load.
- c) the product or part has not been maintained or serviced in accordance with the Instruction manual, the care and hygiene instructions or the manufacturer's recommendations as shown in the maintenance instructions.
- d) Accessories have been used which are not original accessories.
- e) The product or part has been damaged through negligence, accident or incorrect use.
- f) changes/modifications have been made to the product or parts, which deviate from the manufacturer's specifications.
- g) Repairs have been carried out, before our customer service has been informed of the circumstances.

3. To enforce the warranty, please contact your dealer with the exact details of the nature of the difficulty. The product must be repaired by a dealer authorised by Bischoff & Bischoff.

4. For parts, which have been repaired or exchanged within the scope of the warranty, the warranty will be extended to the remaining warranty period for the product in accordance with point 1).

5. For original spare parts which have been fitted at the customer's expense, these will have a 12 months guarantee, following the fitting.

6. If the repair fails after a reasonable time, the customer can, at his discretion, request a reduction, compensation or the termination of the purchase contract.

7. The guarantee is subject to the law of the country in which the product was purchased from Bischoff & Bischoff.

Observe the Bischoff & Bischoff care, warranty, hygiene and maintenance instructions. Your dealer can provide you with this.

Note!

In spite of careful research and editing, incorrect information may have crept into our instruction manuals. We reserve the right to make technical modifications. Illustrations may include accessories. The current version of each instruction manual is ready for download on our website:

www.bischoff-bischoff.com

Non-observance of the operating instructions as well as improper use, maintenance jobs and technical changes or adaptations without the permission of the Bischoff & Bischoff company, lead to guarantee loss and the loss of product liability in general.

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Bischoff & Bischoff France S.A.R.L.