Wichtige Hinweise für Batterien
Important information for Batteries
Información importante para Baterías
Informations importantes pour batteries


DE

EN

ES

FR

Enjoy mobility.

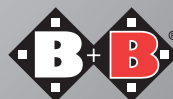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Batteries in electric wheelchairs / scooters

1. Start of operation:

The batteries of your power wheelchair or scooter must be fully charged before first use. To do this, connect the charger supplied with the vehicle. Please refer to the operating instructions supplied separately with the charger for details.

The charging process can take up to 12 hours, depending on the size and charge status of the batteries. As with all future charging processes, the charge status must be read on the charger and not on the wheelchair's battery indicator.

Depending on the lead battery technology (AGM or Gel), the maximum usable battery capacity is only reached after several charging processes (up to 10 complete charging processes). During this time, take particular care not to run the batteries completely empty and to recharge them as fully as possible after each journey.

2. Daily use:

If possible, fully charge the batteries of the electric vehicle before and after every journey.

Familiarise yourself with the function of the battery indicator (see operating instructions for your electric vehicle) and check it regularly while driving.

Plan your journeys so that there is always enough battery charge to enable you to return to the starting point or, if necessary, to the next charging point.

If the battery charge level is very low, an acoustic warning, e.g. a repetitive "double beep", is emitted in addition to the flashing battery indicator, depending on the model.

From this point onwards, the remaining reach of the wheelchair is very short, – a few hundred metres–, then fully charge the batteries immediately! The warning tone is only deactivated after the battery has been fully charged. (See display on the charger).

If the warning tone is still active despite being fully charged, acknowledge the warning tone as follows: with the charger still connected to the wheelchair and switched on, switch the wheelchair on using the on/off button or the driving key and switch it off again after a few seconds.

Please also observe the instructions for use of the supplied charger, which are described on the charger itself and in the operating instructions for the charger. Only use the charger supplied and approved by Bischoff & Bischoff!

Batteries in electric wheelchairs / scooters

3. Storage, replacement, disposal of batteries

If the electric vehicle is taken out of service/stored for a longer period of time, it must be stored fully charged in a dry, frost-free room.

To counteract the self-discharge of the batteries, please charge the electric vehicle at least once a month for at least 24 hours using the charger supplied.

The Gel or AGM batteries used in the wheelchair are maintenance-free over their entire service life. This is highly dependent on individual use and charging habits. To ensure maximum battery life, please follow the recommendations described here.

Only have the batteries replaced and disposed of by qualified specialist personnel from your specialist dealer. Ensure that only batteries approved and tested by Bischoff & Bischoff are used.

4. Safety instructions

Explanation of the warning symbols on the batteries:

- A. Corrosive (battery acid)
- B. No smoking, no naked flames, no sparks
- C. Explosive gas mixture
- D. Recycling
- E. Do not throw in the rubbish
- F. Observe the operating instructions
- G. Wear safety goggles
- H. Do not tip the battery, only use authorised lifting and transport equipment.
- I. Dangerous electrical voltage

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Bischoff & Bischoff GmbH